

How to Create BD Veritor Mini Screen Kits for At-Home Self-Screening

Organizations should split a box of 30 tests into mini screen kits for at-home screening to avoid test wastage.

- Mini screen kits should contain all the supplies needed for up to 4 weeks worth of screening for each participant.

To make the mini screen kits, do the following:

1. The organization should conduct the positive and negative control swabs on each new lot of rapid antigen tests for quality assurance purposes.
2. Collect all the supplies needed for the chosen frequency of screening, including:
 - ✓ packaged swabs
 - ✓ pre-filled reaction tubes
 - ✓ test devices
 - ✓ any written documentation about the at-home screening process
 - ✓ Put supplies into a Ziplock bag or another container and label with the participant's name (if desired).



Note on written documentation:

- Organizations are required to ensure individuals performing at-home screening receive a copy (physical or electronic) of [Provincial Antigen Point-of-Care Testing Guidance](#).
- Any individual conducting at-home screening must also consult the [self-swabbing training resource](#) developed by Ontario Health in collaboration with Public Health Ontario and ensure they have appropriate knowledge, skills and judgement to perform the test.